beurer FT 16 express

DEUTSCH

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ENGLISH

Read the instructions for use carefully and keep them for later use. Make them accessible to other users and note the information they contain.

INCLUDED IN DELIVERY

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- Thermometer • Protective cap • Instructions for use • 1 x 1.5 V LR 41 battery

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

⚠️ WARNING
Indicates a potentially impending danger. If it is not avoided, there is a risk of death or serious injury.
⚡ CAUTION
Indicates a potentially imminent hazard. If it is not avoided, slight or minor injuries may result.
NOTICE
Indicates a potentially harmful situation. If it is not avoided, the device or something in its vicinity may be damaged.
i Product information Note on important information
📖 Observe the instructions Read the instructions before starting work and/or operating devices or machines
 The electronic device must not be disposed of with household waste
 Batteries must not be disposed of with household waste
 Manufacturer
 Date of manufacture
 CE labelling This product satisfies the requirements of the applicable European and national directives.
 Marking to identify the packaging material. A = Material code, B = Material number. 1-7 = Plastics, 20-22 = Paper and cardboard
 Separate the product and packaging elements and dispose of them in accordance with local regulations.
 Protected against solid foreign objects 12.5 mm in diameter and larger, and against the effects of temporary immersion in water
 — Direct current
 Unique device identifier (UDI) Identifier for unique product identification
 REF Item number
 SN Serial number
 MD Medical device
 # Type number
 Type BF applied part
 Temperature limit
 Humidity limitation
 Importer
 Protect from sunlight and heat
 Swiss authorised representative

INTENDED PURPOSE

Intended use

The thermometer is intended exclusively for the orally, rectally or axillary measurement of human body. The thermometer is only intended for private use in a domestic environment and not for commercial use.

Intended users

The thermometer is intended for use in the home healthcare environment, not in professional healthcare facilities. The thermometer can be used on newborns weighing 2.5 kg and above, as well as on infants, toddlers and children under supervision; it can be used for self-treatment by anyone over 12 years of age.

Clinical benefits

The recorded measurements can support in diagnoses and treatments.

Indication

An elevated body temperature is suspected. The user can quickly and easily measure the human body temperature in the anus, oral cavity or armpit with the thermometer and thus determine whether the person concerned has a fever.

Contraindication

- The use of the thermometer on different persons may be inappropriate for certain acute, infectious diseases due to the possible spread of germs, despite the cleaning and/or virus disinfection performed. In individual cases, ask your attending physician.
- Persons with limited mental, sensory or intellectual abilities should be supervised by a person responsible for their safety and should receive instructions from this person on how to use the device.

WARNINGS AND SAFETY NOTES

General notes

⚠️ WARNING

- Keep packaging material away from children. There is a risk of suffocation.
- Keep the device away from children, animals, and vermin.
- The measurements you take are for your information only – they are not a substitute for a medical examination! Discuss your measured values with your doctor and never make your own medical decisions based on them or against medical advice.
- If you obtain unusual temperature results, please contact your GP straight away.

⚡ CAUTION

- If the device has been stored at temperatures between 5 °C or above 40 °C, allow it to acclimatise at an ambient temperature of 5 °C–40 °C for approximately 15 minutes before use.
- Protect the device from impacts, moisture, dirt, severe temperature fluctuations, and direct sunlight.
- Do not bite the thermometer (measuring tip). This can cause damage and injury!
- Check the thermometer for signs of damage or wear before each use. If the thermometer is damaged or worn, it must no longer be used.
- Do not bend the tip more than 90°!

- The thermometer is for the measurement of human body temperature only.
- The thermometer is only intended for the measuring points on the human body that are stated in the instructions for use.
- The minimum duration of measurement until the acoustic signal must always be observed.
- This thermometer features a flexible measuring tip and therefore provides more comfort and safety during measurement, in particular for young children, sleeping people, or people with a reduced level of physical ability.
- The thermometer conducts a self-test when you switch it on. There is no need to check the measurement accuracy.

Notes on handling batteries

⚠️ WARNING

- Risk of explosion Risk of fire!** Failure to comply with the following points can result in personal injury or cause overheating, leakage, venting, or fire. Do not use the device if it is damaged.
- This device contains non-rechargeable batteries which must not be charged.
- Do not throw batteries into a fire.
- Never charge, forcibly discharge, heat, disassemble, open, crush, deform, pierce, or modify batteries.
- Never short-circuit batteries or battery compartment contacts.
- Protect the batteries from direct sunlight, rain, heat, and water.
- Exposure of batteries to an environment with extremely high temperatures or an extremely low air pressure may result in explosion or leakage of flammable liquids and gases.
- Disposal of defective and discharged batteries immediately and properly (see chapter on disposal).
- Do not use modified or damaged batteries.
- Always select the correct battery type.
- Always insert the batteries correctly, taking into account the polarity (+ / -).
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- If fluid from a battery comes into contact with your skin or eyes, wash the affected areas with water and seek medical assistance.
- Choking hazard!** Keep batteries out of the reach of children. Seek medical attention immediately if swallowed. Swallowing them may cause burns, severe internal injuries, and death.
- Never allow children to replace batteries without adult supervision.

⚡ CAUTION

- Store batteries in a well-ventilated, dry, and cool place in a non-conductive container in which the batteries cannot be short-circuited to each other or by other metal objects.
- Keep batteries clean and dry.
- Keep batteries away from water.
- If the device is not going to be used for a long period of time, remove the batteries from the battery compartment.

NOTICE

- Do not use rechargeable batteries.

Notes on electromagnetic compatibility

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The device may not be fully usable in the presence of electromagnetic interference. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner outlined above, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's resistance to electromagnetic interference; this can result in faulty operation.
- Failure to comply with the above could impair the performance of the device.

INFORMATION ABOUT THIS THERMOMETER

Temperature measurement in the anus (rectal)

This measurement method is the most reliable and accurate. Guide the tip of the thermometer carefully 2 to 3 cm into the anus.

Temperature measurement in the oral cavity (oral)

Inserting the tip of the thermometer into one of the two heat pockets under the tongue or next to the root of the tongue.

Temperature measurement under the armpit (axillary)

This measurement method is relatively imprecise, therefore it cannot be recommended from a medical point of view.

Measurement method	Recommended duration of measurement	Mean deviation from rectal deviation
in the rear passage	until the acoustic signal	–
in the oral cavity	until the acoustic signal	–0,4 °C to –1,5 °C (–0,7 °F to –2,7 °F)
under the armpit	min. 5 minutes	–0,7 °C to –2,0 °C (–1,26 °F to –3,6 °F)

USAGE

Briefly press the ON/OFF button to switch on the device. A short acoustic signal confirms that the device is switched on. The thermometer first conducts a self-test for approx. 2 seconds. All segments of the display are visible during this time. The last measured value is then shown. The reference value of 37 °C (98,6 °F) will then appear. Then the measurement symbol "°C" ("°F") flashes and "Lo °C" ("Lo °F") is shown on the display. The thermometer is ready for use. During the measuring process the current temperature value is continuously shown and the "°C" ("°F") symbol flashes. Measuring ends when a stable temperature is reached. An acoustic signal will sound (10 long beeps), the "°C" ("°F") symbol will no longer flash, and the measured temperature will be displayed. While the measured temperature remains below 32 °C (89,6 °F), the measurement symbol "Lo °C" ("°F") will flash and "Lo °C" ("Lo °F") will be shown in the display. Higher values will be displayed directly, e.g. 32,1 °C (89,8 °F). If temperatures of 37,8 °C (100 °F) or higher are measured, a fever indicator will sound (30 short beeps in 10 seconds). If temperatures above 42,9 °C (109,2 °F) are measured, the "Hi °C" ("Hi °F") symbol will appear. The measurement takes much less time than conventional thermometers and only lasts approx. 10 seconds for real measurements. More accurate measurements are achieved by extending the duration of the measurement beyond the acoustic signal. When measuring under the armpits, the recommended duration of measurement should be adjusted to the body temperature. To extend the measurement of the battery life, please switch off the thermometer after taking a measurement by briefly pressing the ON/OFF button. Otherwise the thermometer will automatically switch off after approx. 10 minutes.

MEMORY

The last measured value is automatically saved. Press the ON/OFF button to display this value. The thermometer first carries out a self-test. Next, the last stored measured value is displayed for approx. 2 seconds and then the reference value of 37 °C (98,6 °F) and "Lo" are shown on the display. The last measured value is then shown. The reference value of 37 °C (98,6 °F) will then appear. Then the measurement symbol "°C" ("°F") flashes and "Lo °C" ("Lo °F") is shown on the display. The thermometer is ready for use. During the measuring process the current temperature value is continuously shown and the "°C" ("°F") symbol flashes. Measuring ends when a stable temperature is reached. An acoustic signal will sound (10 long beeps), the "°C" ("°F") symbol will no longer flash, and the measured temperature will be displayed. While the measured temperature remains below 32 °C (89,6 °F), the measurement symbol "Lo °C" ("°F") will flash and "Lo °C" ("Lo °F") will be shown in the display. Higher values will be displayed directly, e.g. 32,1 °C (89,8 °F). If temperatures of 37,8 °C (100 °F) or higher are measured, a fever indicator will sound (30 short beeps in 10 seconds). If temperatures above 42,9 °C (109,2 °F) are measured, the "Hi °C" ("Hi °F") symbol will appear. The measurement takes much less time than conventional thermometers and only lasts approx. 10 seconds for real measurements. More accurate measurements are achieved by extending the duration of the measurement beyond the acoustic signal. When measuring under the armpits, the recommended duration of measurement should be adjusted to the body temperature. To extend the measurement of the battery life, please switch off the thermometer after taking a measurement by briefly pressing the ON/OFF button. Otherwise the thermometer will automatically switch off after approx. 10 minutes.

CHANGING THE UNIT OF MEASUREMENT

You can set the device to display the temperature in degrees Celsius (°C) or degrees Fahrenheit (°F). To set this, press and hold the ON/OFF button for more than 2 seconds when the device is switched on. The unit of measurement used measures. The saved value is displayed in the new unit of measurement.

BATTERIES

Replacing the batteries

The clinical thermometer contains a long-life alkaline-manganese battery (LR 41, 1.5 V) with a battery life of approx. 2000 measurements. The battery needs to be replaced when the battery symbol  at the bottom right edge starts to flash. Pinch the battery compartment lid at the end of the device and pull it firmly back from the body of the device. Remove the old battery and replace it with a battery of the same type. The + symbol on the battery must be facing up. Then place the battery compartment lid back onto the housing.

CLEANING AND MAINTENANCE

Cleaning the device

The device must be cleaned and disinfected before and after each use. The thermometer is waterproof and can be submerged in water or disinfecting solution for the purposes of cleaning or disinfecting without any problems. Clean the thermometer before disinfecting. To do this, wash it with a clean soft cloth to remove any residue. There are two methods of disinfecting the device: Immerse the measuring tip of the device in isopropyl alcohol (70%) for 1

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