DEUTSCH

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ENGLISH

Intended use

The basal thermometer is a special digital thermometer designed to measure the basal temperature in women of childbearing age. The maximum temperature measured with the sensor is saved until the next measurement. Transferring the measurements to the Ovy app enables fertile and non-fertile days in the cycle to be predicted, as well as periods.

Symbols and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and the accessories:

	Attention
	Observe the instructions for use
	Application part, type BF
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Manufacturer
	Protected against foreign objects equal to or greater than 12.5 mm and against temporary immersion in water
	Batch designation
	Permissible storage and transport temperature and humidity

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Important safety notes – retain for future reference.

To make optimum use of all the benefits of the thermometer, read the instructions for use carefully before initial use, retain them for future use and make them available to other users.
 • The thermometer is intended for the measurement of human body temperature only.
 • The thermometer is only designed for the measuring area on the human body stated in the instructions for use.
 • Do not leave children unattended with the device.
 • Check the thermometer for signs of damage or wear before each use. If the thermometer is damaged or worn, it must no longer be used.
 • The minimum duration of measurement until the acoustic signal must always be observed. Discuss the measured values with your doctor.
 • The thermometer contains delicate electronic components. It should therefore be protected against impacts, bending, high temperatures and direct sunlight.
 • This thermometer features a flexible measuring tip and therefore provides more comfort and safety during measurement, in particular for people with a reduced level of consciousness. Do not bend the tip more than 90°.
 • Operation in the vicinity of strong electromagnetic fields, for example next to a mobile phone, may cause malfunctions.
 • The thermometer conducts a self-test when you switch it on. There is no need to check the measurement accuracy.
 • If you obtain unusual temperature results, please contact your doctor straight away.

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Empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries. The codes below are printed on batteries containing harmful substances:

Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.

This battery contains: Pb, Hg.

Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- Choking hazard!** Small children may swallow and choke on batteries or the battery compartment. Store batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
- Risk of explosion!** Do not throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, open or crush the batteries.

Cleaning the device

Clean the basal thermometer before and after each use with a soft cloth and isopropyl alcohol diluted with water or cold soapy water. The device can be immersed in water or a disinfectant solution to clean it. The maximum immersion time is 30 minutes. Do not sterilise the device by boiling, with gas or with an autoclave.

Storage and disposal

Please store the thermometer in the original protective cover when it is not in use. The accuracy of this thermometer has been carefully checked and it has been developed with a long service life in mind. If the device is used for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.

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For environmental reasons, do not dispose of the device in the household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

Error messages

	The measured temperature is below 32.00°C and is therefore outside the measurement range.
	The measured temperature is above 42.99°C and is therefore outside the measurement range.
	Electronic fault in the device. If this occurs repeatedly, please contact the Beurer GmbH customer service team.

Technical data

Type	OT20
Measurement range	32,00 °C - 42,99 °C
Measurement accuracy	± 0,05 °C between 35,50 °C and 42,00 °C at an ambient temperature of 18 °C - 28 °C outside of this measuring and temperature range ±0,1 °C
Display	Liquid crystal display (LCD) with 5 digits
Storage and transportation conditions	Temperature -20 °C +55 °C Rel. humidity 15 % - 95 % 700 - 1060 hPa ambient pressure

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The Ovy app is free for iOS and Android™

The Ovy app enables you to record your entire cycle and keep an eye on it at any time on your smartphone. Ovy is designed for women who want to have children. The app is available to download for free in the Apple App Store and Google Play. The Ovy app is a certified Class I medical device.

How does Ovy work?

The Ovy app works out your cycle automatically, without pen and paper. This includes your period, ovulation and your fertile and non-fertile windows. To do this, take your temperature when you wake up in the morning and enter this value (to two decimal places) in the Ovy app. To make the prediction even more accurate, add other body signals each day. You should ideally start recording on the first day of your period; i.e. the first day of your menstrual cycle. However, you can also start recording on any day in your cycle. It is important that you record when potential external temperature influences such as a fever, consumption of alcohol or a lack of sleep have occurred.

You can also enter your basal temperature, body signals and influencing factors at a later time, but the temperature measurement should have been taken on the relevant day.

The Ovy app is based on a self-learning algorithm. This means that the more often you enter your basal temperature, the more accurately Ovy will be able to predict your fertile days and your next period.

You can also enter your basal temperature, body signals and influencing factors at a later time, but the temperature measurement should have been taken on the relevant day.

The Ovy app is based on a self-learning algorithm. This means that the more often you enter your basal temperature, the more accurately Ovy will be able to predict your fertile days and your next period.

The Ovy app is based on a self-learning algorithm. This means that the more often you enter your basal temperature, the more accurately Ovy will be able to predict your fertile days and your next period.

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